尺寸：260mm*180mm

Designed for veterinary market

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Before performing a blood glucose test, users should read all the information in the instruction manual.

[Product Name]

Generic Name: Eli-VET™ Pet Glucose Test Strips (Glucose Dehydrogenase Method)

[Packaging Specifications]

Model Number: Eli-VET™ Test Strips

50 strips/bottle × 1

[Intended Use]

For in vitro quantitative determination of glucose concentration in peripheral whole blood and venous whole blood of animals (cats and dogs). Suitable for pet owners to monitor blood sugar levels at home or for veterinarians in medical institutions for cats and dogs. Intended only for monitoring blood sugar control in cats and dogs, not for the diagnosis and screening of diabetes in cats and dogs, nor as a basis for adjusting treatment medications.

[Testing Principle]

The detection is based on the principle of enzyme electrodes and dry chemistry technology. Glucose in the blood reacts with glucose dehydrogenase fixed on the test strip, causing electron transfer and generating a current. The magnitude of the current is directly proportional to the glucose concentration.

[Main Components]

Each bottle includes the corresponding number of test strips, and the box also contains a calibration card for inserting into the device. Basic components of the test strips: Glucose Dehydrogenase 0.8IU, 45% Ferricyanide, and approximately 55% non-reactive ingredients. Pet glucose test strips require use with other accessories: pet glucometer, lancing device, lancets, pet glucose control solution.

[Storage Conditions and Validity Period]

1. Store in a dark, airtight space at 4°C(39°F) to 35°C(95°F), valid for 24 months.
2. The production date and expiration date of the product can be found on the label.
3. Test strips should be used within 3 minutes after being removed from the sealed bottle, and immediately in high humidity environments (relative humidity >80%).
4. After the first opening of the sealed bottle, if stored as required, valid for 90 days; cover the bottle cap immediately after removing a test strip.

[Compatible Instruments]

Eli-VET™ Blood Glucose Monitoring System

[Sample Requirements]

1. Fresh peripheral whole blood or venous whole blood from cats or dogs, the collection site should be dry; possible collection sites include the edge of a cat or dog's ear, the side of the paw, a dog's elbow callus, or the inside of the lip.
2. Venous whole blood samples can be fresh heparin anticoagulated whole blood, EDTA anticoagulated whole blood, or potassium oxalate anticoagulated whole blood. It's recommended to use the samples within 10 minutes after collection by professional veterinary personnel. Hemolyzed or coagulated samples can lead to inaccurate test results.
3. Suitable for a hematocrit range of 0%-70%.

[Testing Method]

1. The frequency and timing of testing vary for different animals and should be conducted in accordance with veterinary advice.
2. Before use, ensure that the instrument being used is compatible with the test strips. Refer to the instrument's manual for instructions.
3. Testing Conditions: For optimal test results, please use the test strip under conditions of 5°C(41°F) to 40°C(104°F) and 10% to 80% relative humidity.
4. Testing Method:
Prepare the lancing device (for single animal use only, not to be shared) and lancet (for single use). (Note: Disposable capillary blood collection lancets with safety locks do not require a lancing device; refer to their instructions for specific usage methods.)
Insert the calibration card as per the instrument's manual, open the test strip packaging and insert the test strip into the instrument slot in the correct orientation. The instrument's default setting is dog mode, and the value displayed on the instrument screen should match the dog calibration code value on the test strip packaging.
If the test animal is a cat, after the prompt "please draw blood", press the button on the left side of the instrument to switch to cat mode, and the value displayed on the instrument screen should match the cat calibration code value on the test strip packaging.
Disinfect the blood collection site on the cat or dog with 75% medical alcohol and allow it to dry completely.
Use the lancing device or a disposable capillary blood collection lancet with a safety lock to collect a drop of blood from the cat or dog.
Touch the drop of blood to the test strip's reaction zone at the siphon intake port; the blood will be drawn into the strip.
After a 5-second countdown, the test result will be displayed in mmol/L (mg/dL in USA). Remove/withdraw the test strip, and the instrument will automatically turn off after 5 seconds.

5. System Testing Method:

If you doubt the test results and wish to confirm if the instrument and test strips are functioning properly, you can perform a pet glucose control solution test, following the same method. The quality control result must fall within the control range of the pet glucose control solution.

[Reference Range]

Dog glucose range: 3.3-6.7mmol/L (59-121mg/dL in USA)

Cat glucose range: 3.5-7.5mmol/L (63-135mg/dL in USA)

[Interpretation of Test Results]

1. If the results obtained with this testing system do not match your perceived health condition, please verify if the operation method was correct and retest with a valid test strip. If the situation persists, please consult a doctor promptly. Do not alter medication or other medical advice solely based on the test results without consulting a doctor.
2. If the blood glucose measurement results fall outside the analytical range 1.1-33.3mmol/L (20-600mg/dL), the system will indicate this to the user through symbols: a "Lo" display indicates a blood glucose measurement below the instrument's detection limit 1.1mmol/L (20mg/dL); a "Hi" display indicates a blood glucose measurement above the instrument's detection limit 33.3mmol/L (600mg/dL), refer to the user guide.
3. Regular comparisons with hospital measurements should be made when measuring blood glucose values. If there is a discrepancy between symptoms and test results, follow the guidance of a doctor.

[Limitations of the Testing Method]

1. If the results obtained with this testing system do not match the animal's condition, please verify if the operation method was correct and retest with a valid test strip. If the situation persists, consult a veterinarian promptly. Do not alter medication or other medical advice solely based on the test results without consulting a veterinarian.
2. If the blood glucose measurement results fall outside the analytical range of 1.1-33.3mmol/L (20-600mg/dL), the system will indicate this to the user through symbols: a "Lo" display indicates a blood glucose measurement below the instrument's detection limit of 1.1mmol/L (20mg/dL); a "Hi" display indicates a blood glucose measurement above the instrument's detection limit of 33.3mmol/L (600mg/dL). Refer to the instrument's user manual.
3. Regularly compare measured blood glucose values with those obtained at a pet hospital. If there is a discrepancy between symptoms and test results, follow

the veterinarian's guidance.

[Limitations of the Testing Method]

1. Eli-VET™ Test Strips, Glucose meter, and Control Solution should be used as a set. Do not substitute with products from other brands.
2. Suitable only for whole blood testing, not for serum or plasma samples.
3. Hematocrit should be between 0% and 70%.

[Product Performance Indicators]

1. Appearance: The test strips should be smooth without burrs, and the sample application area should be clean without stains.
2. Quality Control Substance: 95% of the pet glucose test strips' measurements of pet glucose control solution should fall within the control range of the control solution.
3. Inter-Batch Variation: The inter-batch variation of pet glucose test strips from different batches should be ≤15%.
4. Testing System Accuracy

For blood glucose concentrations below 5.55mmol/L (100mg/dL), the system accuracy results for dogs:

Within ±0.28mmol/L (±5mg/dL)	Within ±0.56mmol/L (±10mg/dL)	Within ±0.83mmol/L (±15mg/dL)
9/13 (69.2%)	13/13 (100.0%)	13/13 (100.0%)

For blood glucose concentrations at or above 5.55mmol/L (100mg/dL), the system accuracy results for dogs:

Within ±5%	Within ±10%	Within ±15%
20/39 (51.3%)	39/39 (100.0%)	39/39 (100.0%)

For blood glucose concentrations below 5.55mmol/L (100mg/dL), the system accuracy results for cats:

Within ±0.28mmol/L (±5mg/dL)	Within ±0.56mmol/L (±10mg/dL)	Within ±0.83mmol/L (±15mg/dL)
14/14 (69.2%)	14/14 (100.0%)	14/14 (100.0%)

For blood glucose concentrations at or above 5.55mmol/L (100mg/dL), the system accuracy results for cats:

Within ±5%	Within ±10%	Within ±15%
22/36 (61.1%)	36/36 (100.0%)	36/36 (100.0%)

[Precautions]

1. This product is for in vitro testing only and is for single use. Do not reuse or use strips that have passed their expiration date.
2. If alcohol is used to disinfect the animal's blood collection site during the blood sample collection process, allow the alcohol to dry completely before collecting blood.
3. Ensure a sufficient blood sample volume for the test strip on the first application. Do not reapply blood to the same test strip. The test strip should not be removed and reinserted into the instrument after application.
4. To ensure the best shelf life of the test strips, store them in an environment of 4°C(39°F) to 35°C(95°F). Do not freeze the test strips or store them in places with high temperatures or humidity. Improper storage of the test strips can affect test results.
5. For optimal test results, use the test strip under conditions of 5°C(41°F) to 40°C(104°F) and 10% to 80% relative humidity. Use the test strip within 3 minutes of opening, and use immediately in high humidity environments (relative humidity >80%).
6. After each test, properly dispose of used lancets and pet glucose test strips. Used test strips and lancets are considered medical waste, so please comply with your local medical waste disposal regulations and do not reuse.
7. The product should not be used for definitive diagnosis or medication guidance. Testing should be conducted under the guidance of a professional veterinarian. The interpretation of test results should be determined by a professional veterinarian, not by the user.

Manufactured for:

ELIMEDICAL

ELIMEDICAL, INC.

450 Alden Road, Unit 16, Markham, ON L3R 5H4, CANADA

Importer in USA: ELIMEDICAL-US LLC.

11 Stonelea Drive, Princeton Junction, NJ 08550, USA

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